


# Via<sup>®</sup>

Superior sound quality  
and connectivity



**AUDIBEL<sup>®</sup>**

POWERED BY Starkey Hearing Technologies

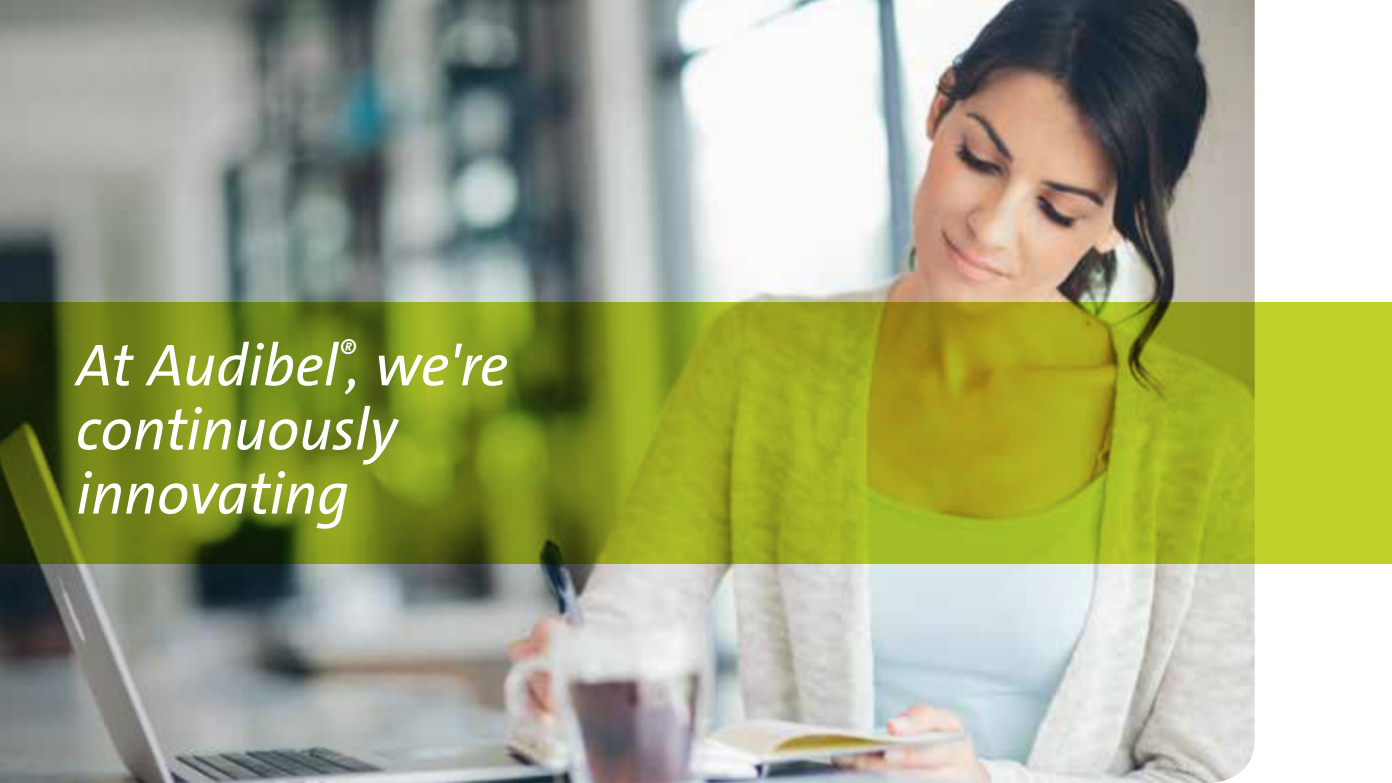

*At Audibel®, we're  
continuously  
innovating*

Audibel is a world leader in the development of innovative hearing aid technology.

Over the past decade, hearing aids have transitioned from single-purpose devices designed to improve audibility for people with hearing loss to “connected” devices that let users stream phone calls, television, music and other audio signals directly from their smartphones to their hearing aids.



**Via®**



# Hearing Reality is a new standard in clarity and comfort

With Via, you'll enjoy superior sound clarity, comfort and universal connectivity. Thanks to a new technology we call **Hearing Reality™**, Via is designed to be effective in even the noisiest environments, comfortable in loud settings, and quiet in quiet settings. It also provides newly enhanced clarity of speech.

Via hearing aids work seamlessly with accessories and your favorite digital devices to help you connect to your world, and to:

- Deliver superior sound quality, listening clarity, more natural hearing and effortless transitions
- Preserve peace and quiet while amplifying the important things, and help you better understand conversations
- Stream phone calls, music and messages directly from your smartphone to your hearing aids
- Offer remote programming so your hearing care professional can make minor adjustments without an office visit

# The connection between health and hearing

Hearing health is directly connected to overall health and well-being. That's been proven, and it means people who are able to hear can engage with the world around them and live better and healthier lives.

Ever-increasing evidence also shows treatment of hearing loss with hearing aids can improve or limit some serious health issues like cognitive decline and dementia.<sup>1</sup> In fact, a 2017 study revealed that treating hearing loss is the single greatest change a person can make to lower their risk for dementia.<sup>2</sup>

Other research supports that it's a change worth making. Adults with moderate to severe hearing loss are three to five times more likely to develop dementia.<sup>3</sup> And adults with hearing loss can experience a 30 to 40 percent faster rate of cognitive decline than those with normal hearing.<sup>4</sup>

<sup>1</sup> Bassuk et al., 1999; Béland et al., 2005; Marioni et al., 2015; Zunzunegui et al. 2003

<sup>2</sup> Lancet Commission, 2017

<sup>3</sup> Lin, F. R., Yaffe, K., Xia, J., Xue, Q., Harris, T. B., Purchase-Helzner, E., Simonsick, E. M., 2013

<sup>4</sup> Johns Hopkins Medicine, 2011



# What making better connections sounds like

Via intuitively connects to your Apple® and Android™ devices via Bluetooth® technology and Audibel's easy-to-use Thrive™ Hearing app.

Thrive offers additional features designed to enhance your listening experience anywhere you go, and help you easily enjoy everything you do.



## Remote Programming

Request adjustments from your hearing care professional who can then make them remotely so you don't need to make an appointment and stop in.



## Personalized Control

Make personalized, real-time adjustments to suit your preferences in different listening situations.



## Phone Calls

Answer your calls with the touch of a button and hear an entire conversation streaming directly to your Via hearing aids via Bluetooth technology.\*



## Remote Control

Using the remote control feature in the Thrive app, you can change volume and switch memories on your hearing aids right from your smartphone.

\*Remote Microphone + is required for Android devices.



### Personalized Memories

You can create multiple custom memories in the Thrive app by using any of the customize options.

You can even geotag memories. A geotagged memory will use GPS and cellular towers to recognize where you are and automatically adjust your Via hearing aids when you enter that space. For instance, a geotagged “coffee shop” memory will automatically activate when you walk into your favorite coffee shop.



### Siri Integration

You can have Siri® read texts and emails directly to your Via hearing aids.\*



### Find My Hearing Aids

Easily locate lost hearing aids using the Find My Hearing Aids feature, with both a location and timestamp. A signal detector sends a stronger or weaker signal based on how close you are to locating your Via hearing aids.



### Auto Experience Manager

Automatically adjusts your Via hearing aids' loudness over time to help you transition to your new hearing experience in the most comfortable way possible.



### Apple Watch Compatible

Use the Thrive app for Apple Watch® to control volume, change memories and mute your hearing aids right from your wrist.\*



### Real-Time Notifications

Notifications for phone calls, texts, emails or other apps can be streamed directly to your Via hearing aids.



### Live Microphone

Simply set your smartphone nearby, turn on the microphone feature and stream conversations directly to your Via hearing aids. You can also record, play back and email audio as it happens or save recordings to listen to later.



### Adaptive Car Mode

With Adaptive Car Mode, Via hearing aids will automatically change to a setting designed to reduce the annoying sounds of driving and enhance your overall driving experience.

# Enjoy universal connectivity with new accessories

Our new hearing aid accessories are powered with wireless connectivity to help users fully engage with the world around them.



## TV

Stream audio from your TV or other electronic audio source directly to your Via hearing aids. It offers excellent sound quality, is easy to use and supports both analog and digital input sources.



## Remote Microphone +

Easily stream from a variety of audio sources. And thanks to Remote Microphone +, Via is the first hearing aid to feature Amazon® Alexa connectivity.



## Remote

Our new remote includes updated features so you can control memory and volume, mute your hearing aids and turn other special features on and off.

# What living better sounds like


## To hear comfortably in challenging environments:

With Hearing Reality, Via is effective in even the noisiest environments, comfortable in loud settings, and quiet in quiet settings. You'll also enjoy newly enhanced speech clarity.



## The convenience of adjustments on the go:

You can easily make adjustments to your Via hearing aids any time, anywhere with the Thrive app.

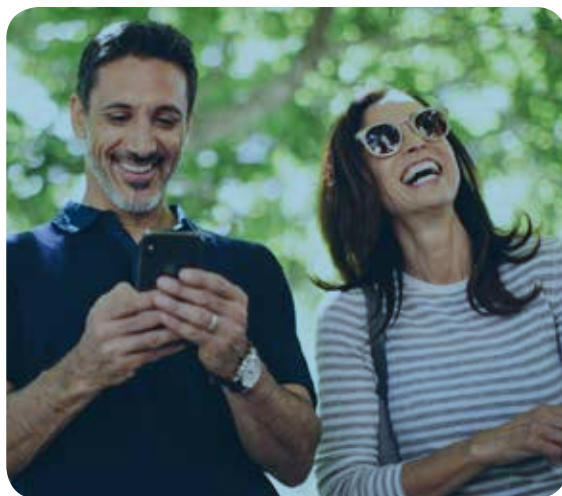

## Better wireless streaming:

New technology provides consistent wireless performance through your smartphone or wireless accessories while streaming cell phones, TV, music and other media.

## Hearing that happens intuitively:

Geotagged memories automatically switch modes when the GPS feature in your smartphone detects you are in a tagged location. For example, a "home" memory activates when you arrive at home.



Hearing that's  
personalized  
for you:

Personalized Control in the Thrive app lets you make adjustments to suit your preferences in different listening environments.

Music the way it was  
meant to be heard:

A whole new way to experience music — adjustable via the Thrive app. Now you can hear every note the way the artists intended — with pure, refined sound quality.

Customizable  
tinnitus relief:

Advanced Multiflex Tinnitus Technology, adjustable via the Thrive app, to bring relief to those who suffer from ringing in the ears.



No buzzing or  
whistling:

Audibel's best-in-class feedback cancellation system provides feedback-free, comfortable listening all day long.

Durable,  
dependable  
hearing aids:


Surface™ NanoShield, our pioneering water, wax and moisture repellent system, to protect and ensure durability and dependability.



# Track brain and body health with Via AI

Another member of the Via product family, **Via AI** is the world's first hearing aid to track brain and body health using integrated sensors and artificial intelligence.

While tracking devices that monitor physical activity already exist, until now there has never been a device that tracks physical activity and cognitive health as measured by hearing aid use in social situations.

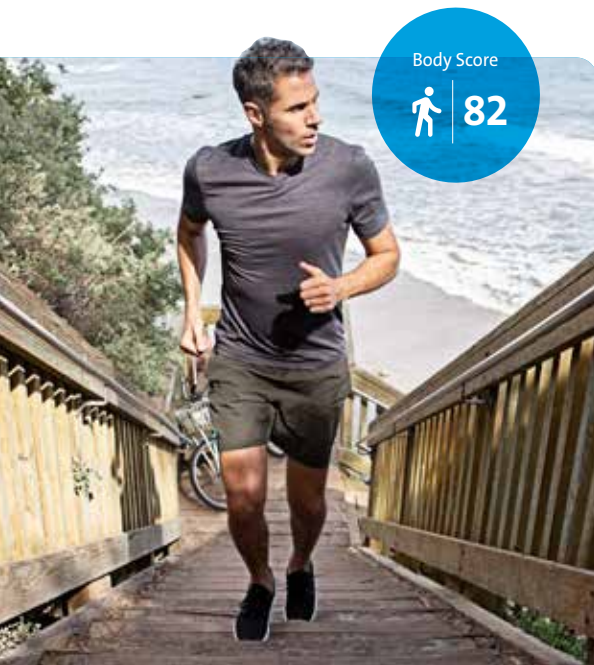

thrive  
+  
Via AI



Most fitness tracking devices use the wrist, but the ear provides far more accurate data — in fact, it's an ideal spot for reliable tracking and Via AI is the first-ever device to take advantage of that.


Compatible with both Android and Apple smartphones, Via AI provides you with a Body Score, a Brain Score and an overall score combining the two, called a Thrive Wellness Score.

Now people of all ages can take a proactive approach to their overall wellness by monitoring their activities.

# The right technology for your lifestyle

Via hearing aids enable you to hear better in all environments — even the most challenging ones.

## Technology Levels

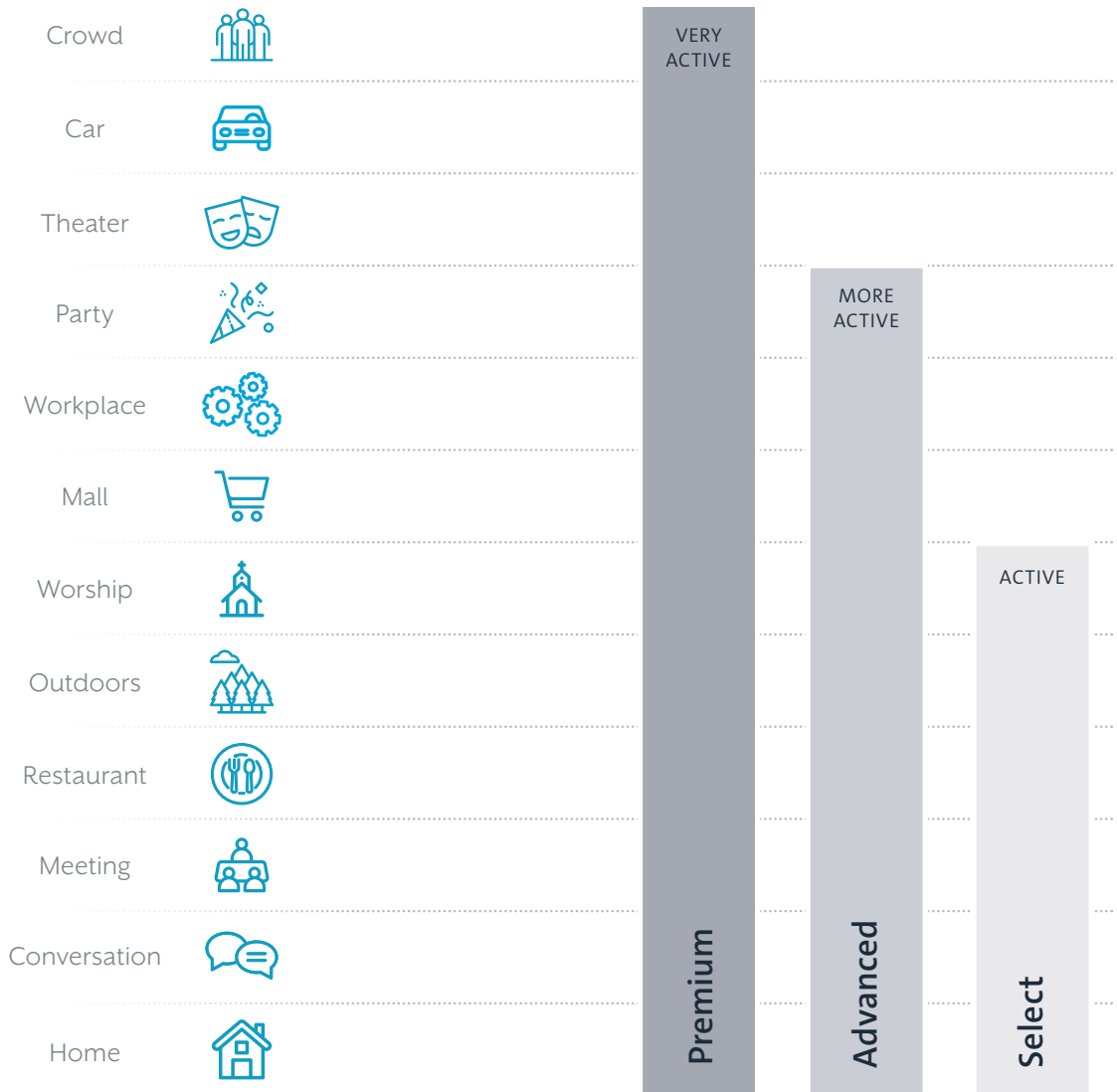

# Choose your style

Pick your favorite color in one of two popular styles.



**RIC 312**  
Receiver-In-Canal



**BTE 13**  
Behind-The-Ear

## Colors



Black



Slate



Sterling



Bright White  
with Sterling



Espresso



Bronze



Champagne

# Changing lives through the gift of hearing

*Our Founder and CEO, William F. Austin, has always held a strong belief that the gift of better hearing results in a fuller and richer life.*

We prove our dedication to this philosophy daily, through our support of Starkey Hearing Foundation.

The gift of hearing opens up new worlds of opportunity. It connects individuals to life and helps them realize they can accomplish more than they ever thought possible. Starkey Hearing Foundation uses hearing as a vehicle to change lives around the world.



## Spreading hope from ear to ear



100+ countries



More than 1 million hearing health care services provided to people in need worldwide



Countless lives changed



## What continuous innovation sounds like

We don't do technology for technology's sake. From the technicians in our labs to the scientists, audiological researchers and every employee at every level, we continually seek ways to better understand the science of hearing loss and its impact on people's lives — so we can continually develop better technologies to improve it.



Hear better. Live better.™

Via, Via AI and Thrive are compatible with iPhone X, iPhone 8 Plus, iPhone 8, iPhone 7 Plus, iPhone 7, iPhone SE, iPhone 6s Plus, iPhone 6s, iPhone 6 Plus, iPhone 6, iPhone 5s, iPhone 5c, iPhone 5, iPad Pro, iPad Air 2, iPad Air, iPad (4th generation), iPad mini 4, iPad mini 3, iPad mini with Retina display, iPad mini, iPod touch (5th generation) and Apple Watch. Use of Apple Watch requires iPhone 5 or later.

Android is a registered trademark of Google, Inc.

Use of the Made for Apple badge means that an accessory has been designed to connect specifically to the Apple product(s) identified in the badge, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards.

Apple, the Apple logo, iPhone, iPod touch, Apple Watch, App Store and Siri are trademarks of Apple, Inc., registered in the U.S. and other countries.

Amazon, Alexa and all related logos are trademarks of Amazon.com, Inc. or its affiliates.

Made for

 iPhone | iPad | Apple Watch



Download on the  
App Store



GET IT ON

Google play

  
**AUDIBEL**<sup>®</sup>

[www.audibel.com](http://www.audibel.com)

---

**Global Headquarters**  
6700 Washington Avenue South  
Eden Prairie, MN 55344-3476  
800.769.2590